

How to care for your eye protection goggles

1

Wipe the inside then outside of your goggles using a cleaning wipe or a clean cloth saturated with a neutral detergent.

2

Wipe the outside of the goggles using a TGA-approved disinfectant wipe or a clean cloth saturated with TGA-approved disinfectant.

3

Wipe the outside of the goggles with clean water to remove any residue.

4

Air dry.

5

Perform hand hygiene.



Scan to
learn more

Produced by Brisbane South PHN

While the Australian Government helped fund this document, it has not reviewed the content and is not responsible for any injury, loss or damage however arising from the use of, or reliance on, the information provided herein.

Care tips

Label your goggles to ensure individual use.

To improve visibility, wipe the goggles with alcohol.

Store your goggles individually in a breathable container.